

ECOSEPTOL FOAM

DISINFECTANT DETERGENT FOAM
ALCOHOL FREE

MD SURFACES TREATMENT



■ PRODUCT ADVANTAGES

- neutral pH solution
- Phosphate free, EDTA free, colorant free, perfume free
- Ready-to-use
- Bottle with foam dispenser
- Compatibility with a wide range of material
- Chloride-free formulation
- Active on enveloped viruses

Fb Franklab®
notre expertise l'Ultra-Propreté



MANUAL
TREATMENT



READY-TO-
USE

CE
0459



ECOSEPTOL FOAM

DISINFECTANT DETERGENT FOAM ALCOHOL FREE

■ INSTRUCTIONS FOR USE

■ 1st use of spray gun :

- Wear gloves and unlock the spray

■ Use (usage):

- 1- Spray uniformly **Ecoseptol FOAM** directly on the surface or on a dry single use wipe
- 2- Spread the solution with the wipe
- 3- Contact time: according to the antimicrobial activity desired (see disinfecting properties)
- 4- Do not rinse*

*The product does not require rinsing, unless specified by the flooring manufacturer's instructions or when in contact with skin or mucous membranes.

- 5- Throw the support

- 6- Repeat whenever necessary.

■ DISINFECTANT PROPERTIES

Activities	Standards	Contact time
Bactericidal	EN 13727	5 min.
	EN 13697	5 min.
	EN 1276	2 min.
	EN 14561	10 min.
Yeasticidal	EN 13624	5 min.
	EN 13697	5 min.
	EN 1650	5 min.
	EN 14562	10 min.
Fungicidal	EN 13624	15 min.
	EN 13697	15 min.
	EN 1650	15 min.
Virucidal	EN 14476 + A2 (Vaccinia Virus)	5 min.
	EN 14476 PRV (hepatitis B virus model) BVDV (hepatitis C virus model)	5 min.
	EN 14476 (Rotavirus)	5 min.
	Sporicidal	EN 13704 Clostridium Difficile EN 17126 Clostridium Difficile

■ PRECAUTIONS OF USE

See SDS (FS230)

■ FEATURES

- Aspect : Flowable liquid
- pH : 6

■ COMPOSITION

- Cationic surfactants
- Non-ionic surfactants
- Chlorinexhidine digluconate
- Sequestrants
- Excipients

■ PACKAGING

- 1 Box of 12 sprays of 750 mL.....ref. **23030**



Follow precautions of use. Read label and product information before use. Class IIb Medical device.