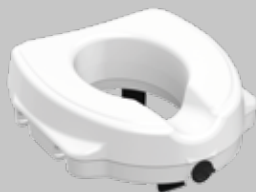
KING RAISED TOILET SEATS

WITH FRONT LOCKING SYSTEM

EN USER MANUAL
DE BENUTZERHANDBUCH
FR MANUEL DE L'UTILISATEUR
ES MANUAL DE USUARIO
IT MANUALE D'USO
PT MANUAL DO USUÁRIO
SL NAVODILA ZA UPORABO
HR KORISNIČKI PRIRUČNIK
HU HASZNÁLATI UTASÍTÁS
RO MANUAL DE UTILIZARE

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WITH FRONT LOCKING SYSTEM

INTENDED USAGE:

The products of the *Raised Toilet Seat with Front Locking System KING* line are designed to increase the height of the toilet bowl to facilitate sitting down and getting up procedures and promote independence and comfort. The front locking system provides the stability of the seat. The models provided with fixed or folding armrests further increase safety and comfort and allow the use of the seat with more autonomy.

INDICATIONS AND CONTRAINDICATIONS:

The products of the Raised Toilet Seat with Front Locking System KING line are aimed at elderly people, wheelchair users, people with reduced mobility of the lumbar, knee, or hip, and all those who might have difficulty in sitting down and getting up from the toilet bowl. The folding armrests make the seat particularly suitable for wheelchair users who resort to lateral transfer to the toilet seat.

SAFETY WARNING AND INDICATIONS:

- The maximum user weight indicated in the fitting instructions must not be exceeded.
- Before every use, be sure the product is in a stable position on the WC and correctly fixed on it by the locking system supplied.
- The armrests must be assembled appropriately, and during use, the moving part must be correctly hooked, as shown in the assembly pictures of the User Manual.
- During the assembly, strictly adhere to what is shown in the User Manual.
- Do not modify the seat and its components in any way.
- Do not use the seat if damaged or altered in any way.
- Be sure the seat's height is such as to make the use of it easy and safe for the user.
- Do not lean on the seat while sitting or standing.
- During use, avoid positions that keep the body weight too much frontward. Wrong positions may lead the user to slip and fall.
- The seat must not be stored outdoors.

MAINTENANCE INFO:

Product maintenance consists of cleaning and sanitation procedures. In order to guarantee safe use and an adequate standard of hygiene, the user should perform these procedures before every use. The user has to make certain of the structural integrity of the device and its components, and the tightness of the screws must be checked. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before proceeding with the most accurate disinfection.
3. Use water or non-aggressive detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth.

Alternatively, the product can be cleaned in domestic dishwashers at the maximum temperature.

REUSE REQUIREMENTS:

The product is suitable for reuse by other users. Before reusing by others, the product must be correctly cleaned and sanitized. Before the transfer of the product to others, the product must be checked by qualified dealers.

CORRECT DISPOSAL PROCEDURE:

Do not dispose of the product in the regular household waste. Ask your local authority beforehand for the correct and environmentally friendly way to dispose of it.

INCIDENT REPORTING:

If there is a serious incident involving this device, contact the manufacturer of this product at info@kingbathrooms.com and the responsible authority in your country.

ITEM CODE	Description	PAGE
KING-12-KN	Raised toilet seat 12 cm/5in King	44
KING-12A-KN	Raised toilet seat 12 cm/5in with fixed armrests King	45
KING-12FA-KN	Raised toilet seat 12 cm/5in with folding armrests King	46-47

ICON	DESCRIPTION
	Made in Italy
	Tools Free Assembly
	Dishwasher Friendly
	Max User Weight
	2 Years Warranty
	Scan the QR CODE to watch the assembly video
	The armrests are meant to help the user to keep sitting in a more resting position and to provide a further support on which they can relieve part of the body weight when sitting down or standing up. However, they must not be used to completely lean on them. For a safe use, do not exceed the max load specifically recommended for the armrests.

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LM Global Design Ltd.